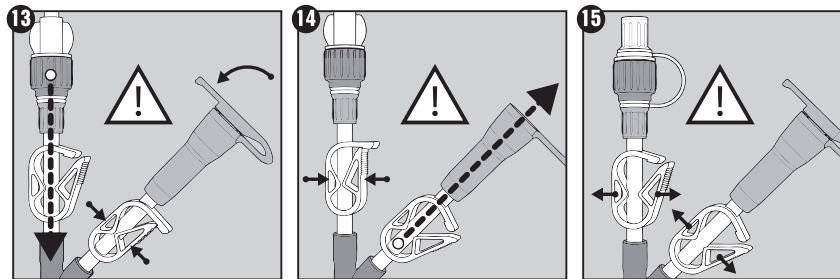
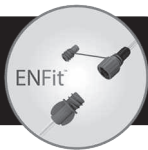
FLOCARE® NASOGASTRIC FEEDING TUBES with SUCTION PORT

EN • Patient Information – Flocare PUR, PURSOFT and Nutrisoft feeding tubes with Guidewire/Suction.

1. What are Flocare Suction tubes and what are they used for?

Nasogastric feeding tubes are intended for delivery of enteral nutrition in the stomach. This product is equipped with an ENFit™ connector intended for enteral nutrition delivery and a suction port intended for gastric suction.

2. Warnings and precautions:

For enteral use only; choking hazard/ loose parts; single patient use only; do not re-use as this may create a risk of contamination and/or altered product performance; no modification of the devices is allowed. Consult your healthcare professional before using the device or in case of any doubt. Any serious incident that has occurred in relation to this device is to be reported to Nutricia and the local competent authority.

3. How is the product used?

Enteral feeding is administered via the “FEEDING” connector. Connect the tube to a giving set or syringe with ENFit connector and begin feeding per healthcare professional’s instructions. Daily care: Vomiting, retching, coughing or accidentally pulling the tube could displace the tube. Should this happen, fluids could go into the lungs instead of the stomach. For this reason, always check the position of the tube via the method indicated by the managing healthcare professional. Contact your healthcare professional in case of doubt and never try to reinsert your feeding tube. Keep the tube free from (feed) debris and clean the connector regularly. Flush the tube with water using an enteral syringe of at least 20mL, before and after the administration of feed or medication and at least every 8 hours to prevent blockage of the tube. Never use excessive force, if there is any resistance, stop the procedure and consult your healthcare professional.

4. How is the suction port used?

The suction port with “△” embossed should NOT be used for enteral nutrition delivery. During feeding the clamp on the suction port should be closed to prevent accidental opening of the cap due to pressure build up in case of an occlusion (figure 13). When using the suction port, the feeding line clamp should be closed to prevent erroneous flow (figure 14). In case no feed is delivered nor suction is taking place, both suction and ENFit™ connectors can be closed with the attached caps and both clamps should remain open as to prevent tube damage (figure 15).

5. What are the possible complications?

Tube blockage, irritation around the nostrils caused by the tube, nausea, vomiting and aspiration can occur. Consult your healthcare professional in case of any observed complication.

6. How to remove and dispose the tube?

It is recommended to use the Flocare® nasogastric feeding tube for <6 weeks to minimize the risk of complications caused by material degradation or clogging within the tube. Removal should always be performed under medical supervision.

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